

SENSORY HUG CANOE

SKU : COS85CAN

Guidelines for Safe Usage

The sensory hug canoe is a therapeutic item and should not be considered as a toy. You should consult your Occupational Therapist or other suitably qualified healthcare professional if the user has any respiratory difficulties or unique physical or developmental requirements.

- **Suitable for ages 5 years and over.**
- **Use under the supervision of a responsible adult at all times.**
- Do not allow the user to lie face down in the canoe due to danger of suffocation.
- The inflatable is NOT designed to be used on water as a floatation device. Under no circumstances should the product be used in water.
- Inspect the product prior to each use.
- Discontinue use immediately if the product shows signs of wear and tear.

Inflation Instructions

- The product should be inflated by a responsible adult. Children should be kept away from the product and pump during inflation.
- On unpacking the product, ensure that it is spread out and that the two side inflation valves are on the outside.
- Inflate the bottom section first, followed by the two side pods.
- Once fully inflated unplug the pump and keep it away from children.

If you have any questions about the safe use of this product please email info@sensorydirect.com or call us 01905 670500