

Information for Use



Comfort Scarf

SKU: SCARCOMF

WARNINGS:



Keep away from fire



Not suitable for children under 3 years



Adult supervision required



Do not use while sleeping



Do not use when exercising

Important:

- Always ensure the wearer can independently remove the Comfort Scarf.
- Check before use, discontinue using if showing signs of wear and tear.
- The Comfort Scarf is not designed to be worn when active (e.g. exercising, running, jumping etc.).
- Ensure the Comfort Scarf is only secured using the Velcro provided.
- The Comfort Scarf is not a substitute for a hard oral chew.

PRODUCT OUTER	PRODUCT INNER
100% Terry Cotton Fleece OEKO-Tex® Certified	65% Polyester / 35% Cotton OEKO-Tex® Certified
Care Instructions Machine washable 30°C. Do not bleach. Air dry or tumble dry on low heat. Please ensure the Velcro hook and loop sides are fastened together before washing. We recommend using a mesh laundry bag.	

Information for Use



Good to know:

- Dimensions - (L) 61cm x (W) 5.5cm / (W) 14.5cm

Good for:



Calming and Relaxing.
Reducing the need
for repetitive sensory
seeking



Improving state of
wellbeing.
Enabling self-regulation



Improving concentration,
focus & attention.
Improving body
awareness



Safe and non-invasive
therapeutic solution

How to use:

- The Comfort Scarf is designed as a safe alternative to mouthing clothing.
- Place the Comfort Scarf around the neck and fasten with the Velcro.
- The shape of the Comfort Scarf allows the wearer to lower their head to the Comfort Scarf to mouth.

Ideal for:



Education or
Home Study



Chill-Out Time



Home or Residential
Setting



Travelling



Sensory Direct (UK) Ltd, Unit 65c, Blackpole Trading Estate West, Worcester, WR3 8TJ, UK.
www.sensorydirect.com