

Information for Use

Hand and Wrist Covers



SKU: HWC0V

WARNINGS:



Adult supervision required



Do not use while sleeping

Important:

- Recommended age 3+.
- Always ensure the user can independently remove the Hand and Wrist Covers.
- Remove the Hand and Wrist Covers at the first signs of discomfort or distress.
- Check before use, discontinue using if showing signs of wear and tear.
- Do not alter, adapt or wash outside of the care instructions.

PRODUCT OUTER

100% Polyester / 100% Polyester Fleece

Care Instructions

Wipe clean with a damp cloth. Do not machine wash. Do not hand wash. Do not use fabric softener or bleach. Air dry. Do not tumble dry. Alternatively steam clean on a low setting. Please ensure the Velcro hook and loop sides are fastened together before cleaning.

Good to know:

- To ensure a comfortable fit, measure across the palm of the hand to find out the size you need.
- Each Hand and Wrist Cover has a wide Velcro fastening for adjustment and an elastic strap to hold the Hand Cover in place.
- Small size - Suitable for palm width of 5cm up to 6cm.
- Medium size - Suitable for palm width of 6.5cm up to 8cm.
- Large size - Suitable for palm width of 8cm up to 9.5cm.

How to Use:

- The Hand and Wrist Covers are designed for those individuals who like to chew on cuffs or bite the back of their hands.
- Adjust the Hand and Wrist Covers using the large strip of Velcro for a comfortable fit.

Manufactured by Sensory Direct (UK) Limited
Unit 65c, Blackpole Trading Estate West, Worcester, WR3 8TJ, UK
☎ 01905 670 500 ✉ info@sensorydirect.com 🌐 www.sensorydirect.com

