

Information for Use

Weighted Neck Wraps



SKU: WNRIN

WARNINGS:



Keep away from fire



Adult supervision required



Do not use while sleeping



Do not microwave



Use one weighted product at a time

Important:

- Recommended age 5+.
- Always ensure the user can independently remove the Neck Wrap.
- Remove the Neck Wrap at the first signs of discomfort or distress.
- Check before use, discontinue using if showing signs of wear and tear.
- Do not alter, adapt or wash outside of the care instructions.

PRODUCT OUTER	PRODUCT INNER	PRODUCT FILLING
100% Polyester Minky Dimple Fleece	100% Polyester	Tourmaline Beads
<p>Care Instructions</p> <p>Machine washable 30°C. Use a gentle wash and low spin cycle. Do not use fabric softener or bleach. Air dry flat. Do not tumble dry or iron. Do not place on heaters or radiators. Alternatively steam clean or wipe clean with a damp cloth.</p>		

Good to know:

- Weight – 1.2kg
- Dimensions – (L) 75cm x (W) 15cm

Good for:



Calming and Relaxing.
Reducing the need
for repetitive sensory
seeking



Improving state of
wellbeing.
Enabling self-regulation



Improving concentration,
focus & attention.
Improving body
awareness



Safe and non-invasive
therapeutic solution

How to Use:

- The Neck Wrap is designed to be draped around the neck or across the shoulders.
- As a guide, the Neck Wrap should be used for approximately 20-40 minutes at a time and then removed for the same period before they are used again. (Consult with an OT for individual guidance).
- Ensure that the user is consistently comfortable and content with the deep pressure they experience.

Ideal for:



Education or
Home Study



Dining



Chill-Out Time



Home or
Residential Setting



Work or
Home Office



Travelling



Made in China

Sensory Direct (UK) Limited

Unit 65c, Blackpole Trading Estate West, Worcester, WR3 8TJ, UK

☎ 01905 670 500 ✉ info@sensorydirect.com 🌐 www.sensorydirect.com