

WEIGHTED THERAPY ANIMALS - Starfish and Turtle

SKU: SFBG222BL and TUBG222GR

WARNINGS:



Keep away from fire



Contains small parts - Choking hazard



Adult supervision required



Do not use while sleeping



Do not microwave



This is not a toy



Use one weighted product at a time

Important:

- Recommended age 5+.
- Always ensure the user can independently remove the Weighted Therapy Animal.
- Remove the Weighted Therapy Animal at the first signs of discomfort or distress.
- Check before use, discontinue using if showing signs of wear and tear.
- Do not alter, adapt or wash outside of the care instructions.

PRODUCT OUTER	PRODUCT FILLING
100% Polyester Fleece	95% Borosilicate Glass Beads 5% Polycotton Stuffing
Care Instructions Surface clean only. Wipe clean with a damp cloth or sponge. Do not machine wash. Store in a dry environment.	

Good to know:

- Weight – 1.5kg
- Dimensions – (L) 42cm x (W) 30cm X (H) 12cm

Good for:



Calming and Relaxing.
Reducing the need for repetitive sensory seeking



Improving state of wellbeing.
Enabling self-regulation



Improving concentration, focus & attention.
Improving body awareness



Safe and non-invasive therapeutic solution

How to use:

- Plan how you wish to introduce the Weighted Therapy Animal to the user. As a guide, the Weighted Therapy Animal should be used for approximately 20-40 minutes at a time and then removed for at least the same period before it is used again. (Consult with an OT for individual guidance).
- Initially we recommend short periods (10-15 minutes) building up the length of use and the occasions you choose to use the Weighted Therapy Animal.
- Ensure that the user is consistently comfortable and content with the deep pressure they experience.

Ideal for:



**UK
CA**

Made in China