

Deep Pressure Vest

SKU: DPV

We recommend consultation with an Occupational Therapist (OT) or suitably qualified person prior to use.

WARNINGS:



Keep away from fire



Adult supervision required



Do not use while sleeping

Important:

- Always ensure the user can independently remove the Deep Pressure Vest.
- Remove the Deep Pressure Vest at the first signs of discomfort or distress.
- Check before use, discontinue using if showing signs of wear and tear.

PRODUCT OUTER

90% Neoprene / 10% Polyester

Care Instructions

Hand wash only. Do not machine wash. Do not use bleach. Do not tumble dry. Do not iron. Hang dry. Please ensure the Velcro hook and loop sides are fastened together before washing. We recommend using a mesh laundry bag.

Good to know:

- The Deep Pressure Vest is available in the following sizes:
 - Small**
 - 1 x body wrap: 65cm x 20cm.
 - 2 x shoulder straps: 40cm x 10cm.
 - Each wrap/strap has 10cm Velcro on inner at each end.
 - Medium**
 - 1 x body wrap: 85cm x 25cm.
 - 2 x shoulder straps: 50cm x 10cm.
 - Each wrap/strap has 10cm Velcro on inner at each end.
 - Large**
 - 1 x body wrap: 110cm x 30cm
 - 2 x shoulder straps: 60cm x 12cm.
 - Each wrap/strap has 10cm Velcro on inner at each end.

Information for Use



Good for:



Calming and Relaxing.
Reducing the need
for repetitive sensory
seeking



Improving state of
wellbeing.
Enabling self-regulation



Improving concentration,
focus & attention.
Improving body
awareness



Safe and non-invasive
therapeutic solution

How to use:

- We recommend the Deep Pressure Vest is worn for periods of 20-30 minutes and then removed for at least the same period before using again. (Consult with an OT for individual guidance).
- The Deep Pressure Vest body wrap should be wrapped around the user's body so that it is a tight fit but comfortable for the user.
- The Deep Pressure Vest shoulder straps should be placed over each shoulder to secure the Deep Pressure Vest in place.
- We recommend the Deep Pressure Vest is worn over a shirt or t-shirt.
- The Deep Pressure Vest can also be worn discreetly under a jumper.

Ideal for:



Education or
Home Study



Dining



Chill-Out Time



Home or
Residential Setting



Work or
Home Office



Travelling

Made in China

Imported by Bing International Ltd, Unit 14, Elgar Business Centre, WR2 6NJ on behalf of
Sensory Direct (UK) Ltd, Unit 65c, Blackpole Trading Estate West, Worcester, WR3 8TJ, UK.
www.sensorydirect.com