

## Portable Massage Cushion

SKU: VSP45BLK

**Please read the Information for Use carefully** before first use. This Information for Use contains necessary safety advice and user instructions. Please retain for future reference.

### WARNINGS:



Keep away  
from fire



Adult  
supervision  
required



Do not use  
during  
pregnancy



Do not use on  
bruised or broken  
skin



Do not  
immerse in  
water



Keep batteries  
away from  
children



This product  
is not a toy

### Important:

- Batteries should be replaced by a responsible adult.
- Periodically inspect batteries for signs of leakage.

### Good to know:

- Recommended age 5+.
- Powered by 4 x AA batteries (not included).
- Dimensions – Width: 33cm; Height: 30cm; Depth: 10cm; Weight: 0.22kg.
- Made from Polycotton and filled with Polyurethane foam for support and comfort.
- On/off switch function (vibrates as soon as it is turned on, not activated by body weight).
- Convenient elastic band at the back of the cushion to assist with positioning.

### How to use:

- Ensure the cushion is turned off before replacing batteries.
- Unzip the back and insert the batteries into the compartment.
- In the event of pain or nausea when using, discontinue use immediately.
- Recommended usage is a maximum of 15 minutes at a time.

### Care and maintenance instructions:

- Remove batteries before cleaning.
- Clean with a lightly damp cloth - outer only.
- Do not machine wash.
- Ensure product is dry before storage.
- Store in a cool, dry environment.

**UK  
CA**

Made in China

Imported by Bing International Ltd, Unit 14, Elgar Business Centre, WR2 6NJ on behalf of  
Sensory Direct (UK) Ltd, Unit 65c, Blackpole Trading Estate West, Worcester, WR3 8TJ, UK.  
[www.sensorydirect.com](http://www.sensorydirect.com)

