

Information for Use

Scooter Board

SKU: ZSB

Please read the Information for Use carefully before first use. Retain this Information for Use for future reference.

WARNINGS:



Adult supervision required



Do not use with more than one child at a time



Do not stand on this product



Do not use on stairs



Do not ride backwards



Do not exceed the weight capacity of this product

Important:

- Age 3 years+
- Maximum weight capacity – 75kg.
- The Scooter Board is intended for INDOOR USE only.
- The Scooter Board is suitable for HARD FLOORS only.

Good to know:

- Dimensions (including handles) – Width: 53cm; Length: 40.5cm; Weight: 1.5kg.
- Integrated contoured handles to help grip and aid stability and balance.
- Suitable for children to sit, kneel or lie on the Scooter Board.

Ten fun things to do with a Scooter Board:

- **Body bowling** - Set up bowling pins at one end of the room, and have the child ride the Scooter Board to knock them over.
- **Fishing** - Scatter magnetic fish around the floor and have the child go find them all.
- **Superman catch** – Get the child to lie on their tummy and propel themselves to catch or balance objects.
- **Rope pull** - Pull the child around with a rope or a hula hoop.
- **Complete a puzzle** - Have the puzzle pieces at one end of the room, and the puzzle board at the other end of the room.
- **Sorting** - Have bean bags or other objects at one end of the room that can be sorted into containers at the other end of the room.
- **Shopping** - Place play food around the room, tell them what they need to find and get.
- **Obstacle course** – Set up cones, pillows or other obstacles and get the child to scoot around them.

Information for Use



- **Balancing** - Place cones around the room and give the child ping pong balls that they must place on top of the cones.
- **Follow the path** - Create a path using masking tape on the floor and get the child to follow it – if you want to be more adventurous why not try creating a maze!

Care and maintenance:

- **Board maintenance** - Wipe clean with water and a soft cloth. Do not use corrosive cleaning agents (such as disinfectant, alcohol) to prevent material aging. Avoid exposure to sunlight and high-temperature environments to prevent deformation and cracking.
- **Wheel care** – Clean the wheels with a small brush to prevent debris from getting stuck in the bearings and affecting rotation. Avoid long-term sliding on rough surfaces to reduce wheel wear.
- **Inspection** – Regularly check the Scooter Board to ensure the wheel mountings are secure. If not, tighten as required, to prevent detachment.
- **Storage requirements** - When not in use, store the Scooter Board in a dry, well-ventilated area away from damp environments to prevent bearing rust and board mildew. To keep the Scooter Board flat and even, do not stack heavy objects on the Scooter Board.



Made in China

Imported by Bing International Ltd, Unit 14, Elgar Business Centre, WR2 6NJ on behalf of
Sensory Direct (UK) Ltd, Unit 65c, Blackpole Trading Estate West, Worcester, WR3 8TJ, UK.
www.sensorydirect.com