

## Weighted Therapy Turtle

SKU: TUBG222GR

Please read the Information for Use carefully before first use. Retain this Information for Use for future reference.

### WARNINGS:



Keep away from fire



Contains small parts & stuffing - choking hazard



Adult supervision required



Do not use while sleeping



Do not microwave



This product is not a toy



Use one weighted product at a time

### Important:

- Recommended age 5+.
- Always ensure the user can independently remove the Weighted Therapy Turtle.
- Remove the Weighted Therapy Turtle at the first signs of discomfort or distress.
- Check before use, discontinue using if showing signs of wear and tear.
- Do not alter, adapt or wash outside of the care instructions.

### Good to know:

- Made from 100% polyester fleece.
- Filled with 95% borosilicate glass beads and 5% polycotton stuffing.
- Weight – 1.5kg
- Dimensions – (W) 30cm x (L) 42cm X (H) 12cm

### Good for:



Calming and relaxing.  
Reducing the need for repetitive sensory seeking.



Improving state of wellbeing.  
Enabling self-regulation.



Improving concentration, focus & attention.  
Improving body awareness.



Safe and non-invasive therapeutic solution.

## How to use:

- Plan how you wish to introduce the Weighted Therapy Turtle to the user.
- As a guide, the Weighted Therapy Turtle should be used for approximately 20-40 minutes at a time and then removed for at least the same period before it is used again. (Consult with an OT for individual guidance).
- Initially we recommend short periods (10-15 minutes) building up the length of use and the occasions you choose to use the Weighted Therapy Turtle.
- Ensure that the user is consistently comfortable and content with the deep pressure they experience.

## Ideal for:



Education or  
Home Study



Dining



Chill-Out Time



Home or  
Residential Setting



Travelling

## Care Instructions:

- Surface clean only.
- Wipe clean with a damp cloth or sponge.
- Do not machine wash.
- Store in a dry environment.

**UK  
CA**

Made in China

Imported by Bing International Ltd, Unit 14, Elgar Business Centre, WR2 6NJ on behalf of  
Sensory Direct (UK) Ltd, Unit 65c, Blackpole Trading Estate West, Worcester, WR3 8TJ, UK.  
[www.sensorydirect.com](http://www.sensorydirect.com)